

clinell®

www.clinell.com.au

stronger
cleaning power
- ultra low smear

CUTTING EDGE TECHNOLOGY COMBINED WITH EASE OF USE TO CREATE AN INCREDIBLY EFFECTIVE FORMULA

Optimum cleaning

Even stronger cleaning power, developed specifically for use in healthcare environments.

Convenient and effective

Disposable wipes that are ideal for general cleaning and damp-dusting.

Multi-surface cleaning

Can be used on both surfaces and equipment.

Alcohol & disinfectant free

Safe to use and utilise the same technology found in many shampoos and skin care products.

DETERGENT WIPES

Multi-surface wipes for general cleaning and damp dusting. They leave no smear whilst providing a superior level of cleaning power. Suitable for pre-cleaning invasive devices.



OPTIMUM CLEANING

Clinell Detergent Wipes are unlike other detergent wipes. Our detergent formula is incredibly effective and environmentally friendly to provide optimum, multi-surface cleaning and damp dusting in healthcare environments. Many competitors' detergent products are oil and phosphate based which leaves smears and residue.

Clinell Detergent Wipes use compounds that lower the surface tension of a liquid and act as detergents. These are derived from renewable and sustainable plant sources such as sugar cane, rather than non-sustainable oil.

The surfactant molecules are designed for minimum smear and excellent cleaning power.

The efficiency of these molecules also means less surfactant is necessary for a given task, contributing to minimal residue and smear.

Containing no alcohol or disinfectant, they are safe to use and utilise the same technology found in many shampoos and skin care products.

RANGE OF USES

SURFACES


Mattresses, bed frames, table tops, door handles, desks, chairs, ledges, examination couches etc.

EQUIPMENT


Telephones, keyboards and accessories, wheelchairs, trolleys, walking frames etc.



Wearing the recommended PPE, remove one wipe from the pack.

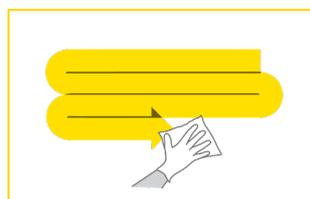

Working from clean to dirty, wipe in an 'S' shaped pattern, taking care not to go over the same area twice.

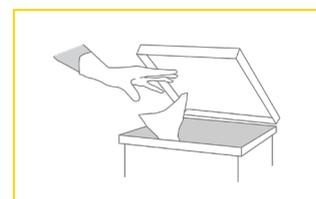

Change wipe if it becomes dry or soiled and discard. Dry the surface with a clean dry paper towel or cloth.

PRODUCT	UNIT OF ISSUE	ORDER CODE
Detergent Wipes Pack 215	Pack of 215	CDW215AUS
White Wall Mounted Dispenser	(for CDW215AUS)	CDWDW



ALWAYS FOLLOW EQUIPMENT MANUFACTURER'S CLEANING PROCEDURES AND GUIDELINES.

GHA230106

GAMA Healthcare Australia Pty Ltd.,
Suite 1, 33-37 Duerdin Street, Notting Hill, VIC 3168, Australia.
T: +61 (0)3 9769 6600 E: info@gamahealthcare.com.au
www.gamahealthcare.com.au

gama
healthcare