

DRAIN DISINFECTANT

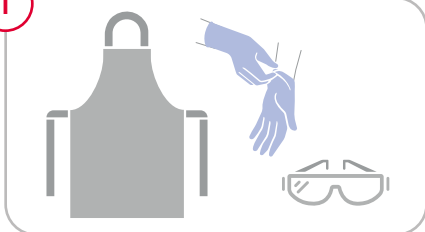
Clinell Drain Disinfectant kills, removes and prevents formation of bacterial biofilms, including *Pseudomonas aeruginosa*, *Escherichia coli* (E. coli) and *Staphylococcus aureus*.

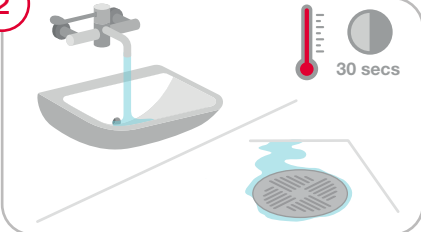
For use in sinks, showers and drains to eradicate persistent pathogens.

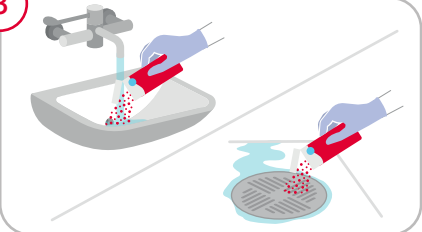


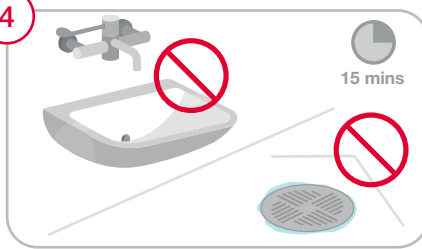
Clinell Drain Disinfectant
(24 x 30g - CSDD24AUS)

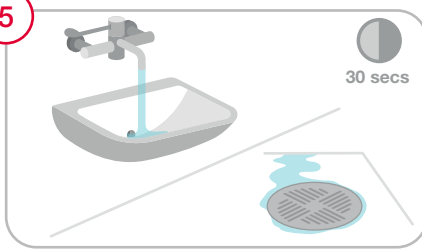
INSTRUCTIONS FOR USE:

- 

Wear appropriate PPE according to your local policy.
- 

Turn on the hot tap/shower for 30 seconds.
- 

Pour entire contents of the sachet into the running water near the drain. Turn off tap/shower immediately.
- 

Wait for at least 15 mins.
DO NOT USE SINK/SHOWER DURING THIS TIME.
- 

Turn on the tap for 30 seconds to flush the drain before using sink/shower.

MINIMUM OF 15 MINS CONTACT TIME IS REQUIRED FOR FULL EFFICACY

For more information, please contact the Infection Prevention and Control Team.

Use biocides safely. Always read the label and product information before use.

GHA230069

clinell®

+61 (0)3 9769 6600 | info@gamahealthcare.com.au

Full product information at:
gamahealthcare.com.au