

SPORICIDAL WIPES

Clinell Sporcidal Wipes are activated by water and are designed for use on blood and bodily fluid spills and where *C. difficile* is suspected or proven.

To use on commodes, bed pans and all hard surfaces.

If unsure please contact the Infection Prevention and Control Manager.




Clinell Sporcidal Wipes
(Pack of 25 - CS25AUS)

SURFACES/EQUIPMENT*

- Commodes
- Toilet facilities
- Raised toilet seats
- Door handles
- Mattresses
- Bed frames
- Bed rails
- Bedside surfaces
- Patient equipment

INSTRUCTIONS FOR USE:

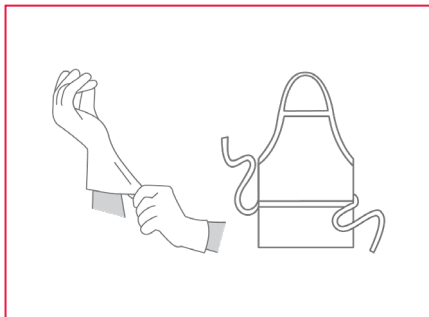


Wear recommended PPE.




Remove wipe from packet.



Activate the wipe with water under a tap, or soak in a bucket.

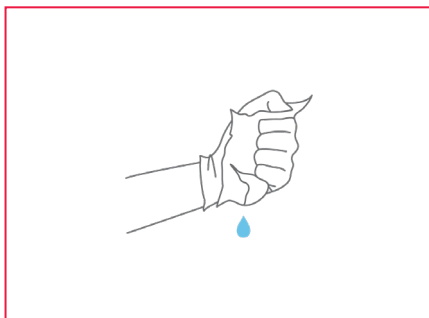

Squeeze wipe over sink or bucket to remove excess water.

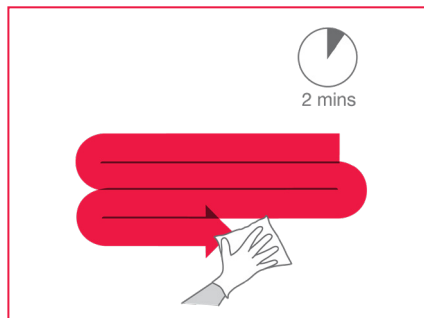

Working from top to bottom, clean to dirty, wipe in an 'S' shaped pattern, taking care not to go over the same area twice.



Dispose of wipe, gloves and apron as hazardous waste. Wash hands with soap and water.

DO NOT FLUSH DOWN TOILETS!

* Please ensure that you follow the manufacturer's guidelines.

GHA190025

clinell®

+61 (0)3 9769 6600 | info@gamahealthcare.com.au

Full product information at:
www.clinell.com.au