



Stop bleeding fast

Topical Haemostatic Dressings

Product Presentation  
& Training




# Regenerated Cellulose

- Carboxymethyl Cellulose Sodium
- **NO** Additives, chemicals, foreign bodies, allergens or by-products
- **Breaks down** to polysaccharides and water
- **Sterile**
- **No adverse affects**



# Features

**Fast** help to control bleeding

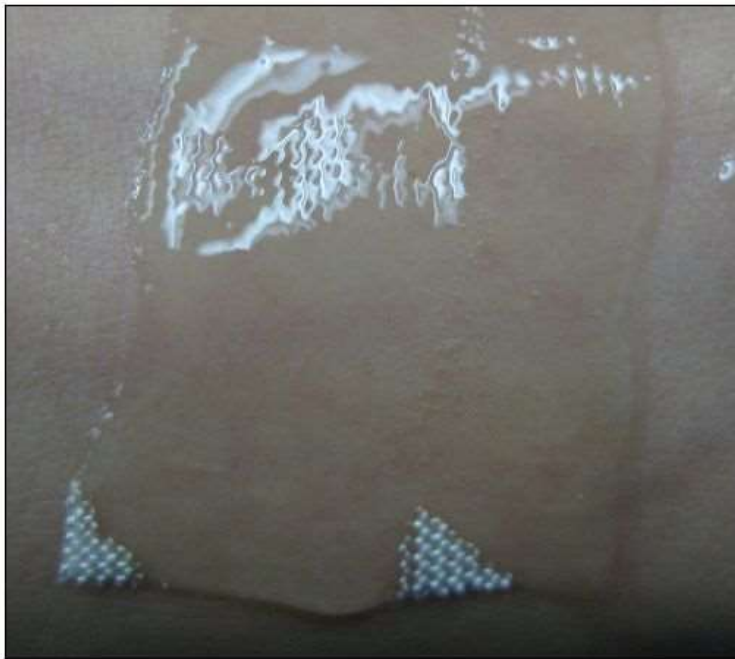

- Reduce incidence of dry socket
- Bacteriostatic properties
- Easy to use
- No need to remove
- No debris
- Safe to ingest
- No adverse effects
- Effective where impaired coagulation/liver function
- Cost effective per use

# Indications for Use

- Disseminated hemorrhage
- Capillary and small vessels hemorrhage
- Trauma and topical surgery
- Patients on medications that impair coagulation
- Patients with impaired liver function
- Not indicated for use in the presence of infection

# Applications for **BLEEDSolv™**

## DENTISTRY & DENTAL SURGERY

- Do not use if package is compromised
- **Remove from sterile packaging**






- Using **dry** gloves/equipment, press product **to source** of bleeding
- Product activates in contact with blood
- Apply pressure / pack with plain gauze / bite down





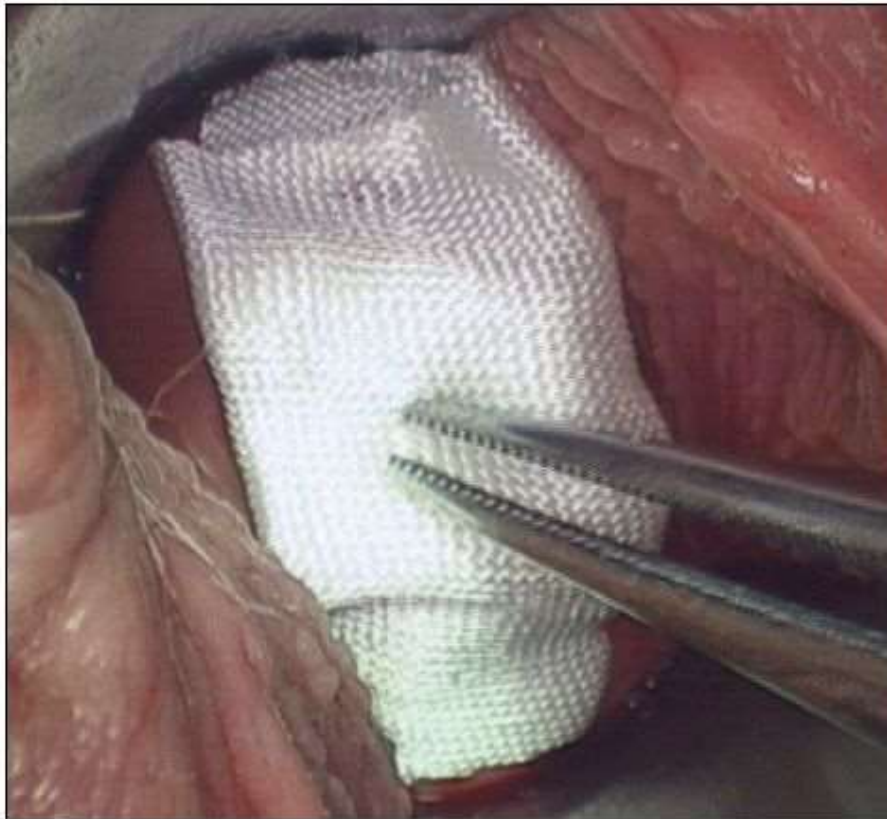


- When applied to deep wounds or significant bleeding:
  - product can be rolled before application
  - press in to the source of haemorrhage

- If needed, after haemostasis, excess product can be washed away
  - Use gentle flushing with saline, liquids
  - No disturbance of clot
  - Wound may be closed without further debridging



# Extraction

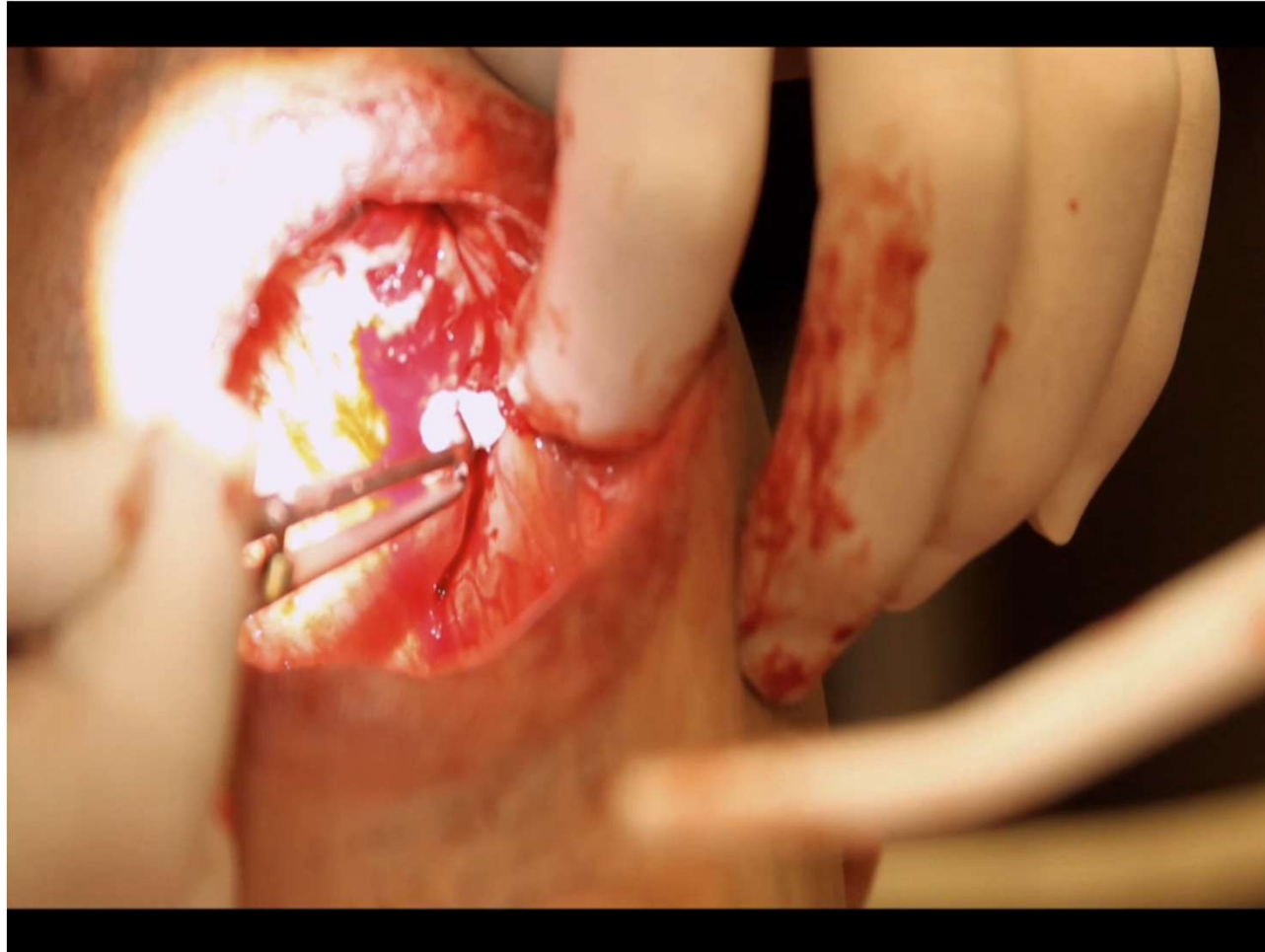

- ◉ Control of bleeding
- ◉ Good adherence
- ◉ Ready for imaging
- ◉ Less time in chair

- Use **dry** equipment
- Can cut to size
- Roll, fold or layer to suit wound
- **Layer** into **sockets**, do not pack
- Flush excess after haemostasis
- Safe for patients to take home
- Do not use if pack is damaged



## TIPS





# Sizes / Applications

Application	Product Code	Sizes
<b>Minor</b>	FAHG: 55, 51, 11	50mm x 50mm 50mm x 100mm 100mm x 100mm
<b>Moderate</b> Consider area, number of damaged capillaries and volume	FAHG: 11, 2020, MT	100 x 100mm, 200mm x 200mm, 90mm x 750mm
<b>Severe</b> Consider tourniquet Consider volume and location	FAHG: 2020, MT	200mm, 90mm x 750mm
<b>Specialty</b> <sup>1</sup> Secure the tampon strings	FADG, FANG	19mm x 19mm blister pack Coated PVC tampon <sup>1</sup>