

SAFE CHECK

INSTRUCTIONS FOR USE – STEAM CHEMICAL INDICATOR Type 6 (Emulating)

Product Description:

Our SAFECHECK Chemical Indicator Steam Type 6 products are Type 6 indicators designed for use in pre-vacuum steam sterilizers operating at 134°C (270°F) for 3,5/5/7 minutes or 121°C for 15 minutes. When used in accordance with the instructions, chemical indicator strips provide instant visual information that sterilization conditions have been met at their location in the sterilizer chamber. When exposed to steam sterilization conditions at 134°C (270°F) for 3.5, 5, or 7 minutes and at 121°C for 15 minutes, the indicator ink will change from its initial color to the reference color. Our products do not return to their original color for 6 months from the sterilization process and can be used for permanent recording. Does not contain lead and heavy metals.

LOT and expiry date information can be found on the product label. The shelf life of our product is 3 years.

Our products meet the performance specifications stated in ISO 11140-1 "Sterilization of health care products - Chemical indicators - Part 1: General specifications" for Class 6 Process Indicators.

Instruction:

- Place at least one indicator strips for each sterilization pack. Placing the strip where the steam reaches the hardest will help you get better results.
- Place the indicator strip in the sterilization device, operate the device as specified according to the operation instruction and the product code in the Packaging and Product Information.
- After a full cycle, remove the indicator strip and observe the color changes.
- If the indicator color reaches the reference color or is darker, the cycle is successful.
- If the indicator color is different than Color Conversion Chart, the cycle is insufficient and the medical devices should be re-sterilized with new indicator strips.

Color Conversion Chart	
Before Sterilization	After Sterilization
	
	

Ozdent Pty Ltd

Unit 12, 6 Gladstone Road, Castle Hill NSW 2154, Sydney, Australia.

Tel +02 8853 5777 www.ozdent.com

Warnings and Precautions:

- The medical device is for single use only. Do not reuse.
- Check the product before using it. Do not use the product that is deformed or has a color change in the indicator color.
- Do not use expired products.

Storage, Storing and Disposal Information:

- Do not expose to direct sunlight and keep it dry.
- Between 0°C and 40°C and Max. Store at 70% RH humidity.
- Store in its original packaging.
- Dispose of in accordance with the place of use and local regulations.

Symbols and their meanings on the product label and user manual:

Symbols	Meaning	Symbols	Meaning	Symbols	Meaning	Symbols	Meaning
	Manufacturer Information		Suitable for steam sterilization		Production Date (Month/Year)		Single Use Only
	Product Lot Information		Max. 70% RH		Expiration date		CE Symbol
	Reference No		Metal free		Unique Device ID		Medical Device
	Do not expose to sunlight		Keep Dry		Lead free		Store between 0°C and 40°C

Ozdent Pty Ltd

Unit 12, 6 Gladstone Road, Castle Hill NSW 2154, Sydney, Australia.

Tel +02 8853 5777 www.ozdent.com

Packaging and Product Information:

Product Code	Product Description	Package Contents	Inner Box Quantity
60.103	Chemical Indicator Steam Type 6 3,5 min	250 strip	80 packs
60.105	Chemical Indicator Steam Type 6 5 min	250 strip	80 packs
60.107	Chemical Indicator Steam Type 6 7 min	250 strip	80 packs
60.113	Chemical Indicator Steam Type 6 3,5 min Adhesive	250 strip	80 packs
60.115	Chemical Indicator Steam Type 6 5 min Adhesive	250 strip	80 packs
60.117	Chemical Indicator Steam Type 6 7 min Adhesive	250 strip	80 packs
60.203	Chemical Indicator Steam Type 6 3,5 min Dual	500 strip	60 packs
60.205	Chemical Indicator Steam Type 6 5 min Dual	500 strip	60 packs
60.207	Chemical Indicator Steam Type 6 7 min Dual	500 strip	60 packs
60.213	Chemical Indicator Steam Type 6 3,5 min Adhesive Dual	500 strip	60 packs
60.215	Chemical Indicator Steam Type 6 5 min Adhesive Dual	500 strip	60 packs
60.217	Chemical Indicator Steam Type 6 7 min Adhesive Dual	500 strip	60 packs

Ozdent Pty Ltd

Unit 12, 6 Gladstone Road, Castle Hill NSW 2154, Sydney, Australia.

Tel +02 8853 5777 www.ozdent.com