


COINFYCARE

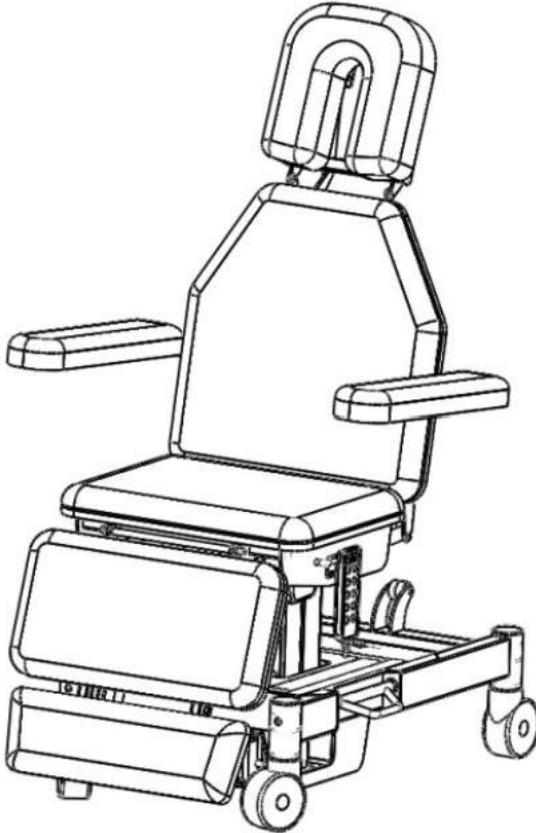

ELT35
Procedure Chair
USER GUIDE

SAFETY PRECAUTIONS

1. During the adjustment of the table do not put any part of the body into the gap of any moving parts.
2. Please operate with both hands when adjusting the angle of back, foot or head sections.
3. When the patient is on the table it is not recommended to adjust the angle of the table top.
4. Before using the product ensure to check the stability of the table frame.
5. Before connecting to the power supply please check whether the incoming cord to the controller is intact
6. A regular inspection (at least once a year) is needed for the treatment table.
7. Do not exceed the table weight capacity of 250 kg. The safe working load of the arm rest is 15 kg
8. Always clean the table after each use with a soft cloth dampened with water and a mild antibacterial detergent. Do not use abrasive cleaners.
9. Disconnect the table from the power source before attempting any maintenance, installation, removal or replacement procedures to prevent electrical shock and possible damage to the table.
10. Please DO NOT make any changes to the table. Replacement parts or labor can be provided by CoinfyCare, the selling dealer or a service technician certified by the manufacturer.

CONTENTS OF CARTON

Electrical Procedure Chair: 1 unit
User Guide: 1 unit

NORMAL OPERATING CONDITIONS

Temperature: 5°C-40 °C
Relative humidity: 30%-75%
Air pressure range: 860-1060hpa
Power supply: AC100-240V 50/60Hz 350VA

SCOPE OF APPLICATION

Hospital orthopedics department, rehabilitation department, chiropractic department, sports injury rehabilitation center, general practice, skin cancer clinics, wound care, aged care, day hospital, surgical procedures.

REGULATION AND PRODUCT STANDARDS

This table complies with the European regulation of medical devices (EU)2017/745 and with the following standards:

- EN 60601-1-2:2006+A1:2013+AC:2014 Medical electrical equipment. Part 1: General Requirements for Basic Safety and Essential Performance.
- IEC/EN 60601-2-52:2009+A1:2015 clause 201.9.8.3.2 Medical electrical equipment. Part 2-52: Particular requirements for basic safety and essential operation of medical tables.
- EN 60601-1-6:2010+A1:2015 Medical electrical equipment. Part 1-6: General Requirements for basic safety and essential performance. Collateral standard: Suitability for use
- EN 60601-1-2:2015 Medical electrical equipment. Part 1-2: General requirements for basic safety and essential performance. Collateral standard: Electromagnetic disturbances. Requirements and tests.
- EN ISO 13485:2016 Medical devices -Quality management systems- Requirements for regulatory purposes (ISO 13485:2016)
DIN EN ISO 13485:2016

PRECAUTIONS AND SAFETY INDICATIONS

CoinfyCare is not liable for any material or personal damage due to misuse or incorrect use of this table or for any purpose other than described in this user guide.

Inspect the condition of the table before using it for the first time and on a recurring basis as indicated in the "MAINTENANCE" section.

Keep table out of high moisture environments. Do not transport the table over rough or outdoor surfaces. The caster system could become damaged and inoperable.

This table should be operated, transported and stored in temperatures between 5°C to +40°C, with max humidity of 75%.

DO NOT operate this table in an environment where other devices are being used that intentionally radiate electromagnetic energy in an unshielded manner. Portable and mobile communications equipment can affect Medical Electrical Equipment

This table generates, uses and can radiate radio frequency energy and if not installed and used in accordance with the instructions may cause harmful interference to other devices in the vicinity. There is no guarantee that interference will not occur in a particular installation. Harmful interference to other devices can be determined by turning this table on and off. Try to correct the interference using one or more of the following: reorient or relocate the receiving device, increase the separation between the equipment, connect the table to an outlet on a different circuit from that which the other device(s) are connected or consult CoinfyCare for help.

Unplug the table from the power supply before starting cleaning or any types of maintenance operations.

The table should be kept out of the reach of children when there is no adult supervision.

The table is designed for discontinuous interval operation so the continuous running time of the motor should never exceed 2 minutes. Should this happen, damage may occur due to overheating of the electronics.

Always use original accessories and spare parts provided by your dealer.

Always cover the table with a towel, disposable paper or a cover and change it for each user thus ensuring correct hygiene and extending the useful life of the upholstery.

Only connect the table to a properly installed and grounded power outlet. The local mains voltage must correspond to the technical data of the device.

The power outlet should be accessible in case the table needs to be quickly unplugged at any given time. Always pull the plug and not the mains cable.

Unplug the table from the power supply before starting cleaning or any types of maintenance operations.

TECHNICAL SPECIFICATIONS

Product dimensions

Length: 1950mm

Width: 900mm(seat width 600mm)

Lifting range of table surface: 590-890mm

If the electrical supply cable is defective, disconnect the table immediately and inform your dealer or a specialized technician.

Position the power cord properly so that it cannot become a tripping hazard.

Unplug the power cord from the mains outlet if the table is not going to be used for a long period of time.

If you detect any abnormality in the operation of the table consult your dealer or a specialized technician.

Never manipulate any electrical part, open or repair any part of the table.

Only your dealer or a specialized technician should repair the table.

Never touch the table or the power cord with wet hands.

Never place your hands or any other part of your body in any hole or void on the table while it is in operation.

Make sure there are no objects in the space under the table that could cause collisions or entrapment during operation.

Make sure that your feet are not resting on the peripheral foot bar, foot or hand controller during treatment and especially when sitting near the table. This could create a dangerous situation where your leg is caught between the section and the peripheral foot bar or floor surface.

Take into account the weight distribution on the table when working. The weight must be distributed along the table without concentrating on its ends to maintain the correct stability.

You should not sit on the head and foot sections. If you do there is a danger of tipping and table damage.

PLACEMENT IN THE WORKPLACE

The table has four casters to facilitate movement to the place where it will be used.

OPERATION

Connect the table to an electrical outlet using the power cord.

Moving and adjusting the table

The table bottom frame is equipped with four central lock casters.

Empty the head of the table and make sure that the casters at the rear of the table are active (Figure 1). The foot bar is lifted up to indicate the unlocked state of the casters.

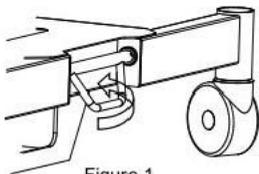


Figure 1

After moving please confirm whether the four casters are in full contact with the ground otherwise step down on the foot bar to lock the casters (Figure 2).

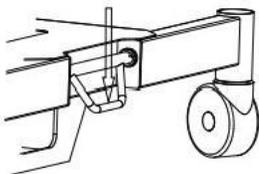


Figure 2

Head angle adjustment

●As shown (Figure 3) pull the control lever (in the direction shown by the arrow) to adjust the angle of the head rest.

When adjusted to the desired angle release the control lever to be parallel to the chair surface and the head rest angle will be fixed.

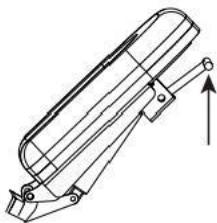


Figure 3

Hand controller functions

The hand controller is for adjusting height, seat tilt, angle of the backrest & leg section of the chair.

Before adjusting any section make sure there are no objects in the space below the table that could cause a collision or entrapment

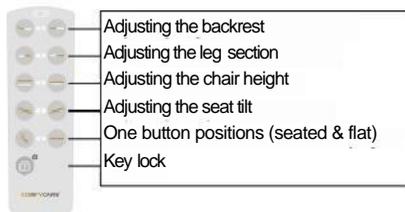


Figure 4

● Press button on line 1 of the hand controller to adjust the backrest of the table upward and downward (Figure 5). When the required angle is reached release the button and the angle of the backrest will be fixed automatically.

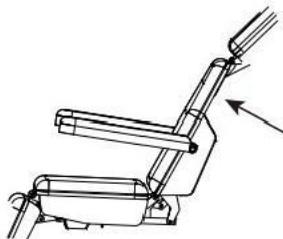


Figure 5

● Press button on line 2 of the hand controller to adjust the leg section of the table upward and downward (Figure 6). When the required angle is reached release the button and the angle of leg section will be fixed automatically.

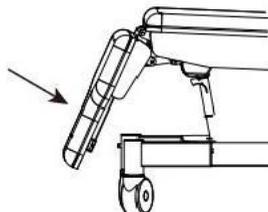


Figure 6

- Press button on line 3 of the hand controller to adjust the height of the table (Figure 7). Adjustable height range is 590mm-890mm.

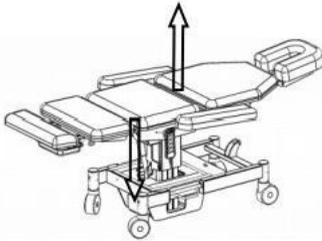


Figure 7

- Press button on line 5 of the hand controller to achieve a one button lay flat position (Figure 10).

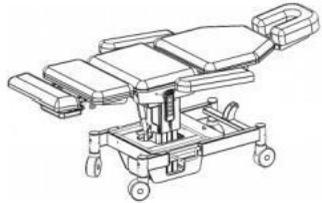


Figure 10

- Press button on line 4 of the hand controller to adjust the seat tilt angle of the chair (Figure 8). When the required seat tilt angle is reached release the button and the tilt angle will be fixed automatically.

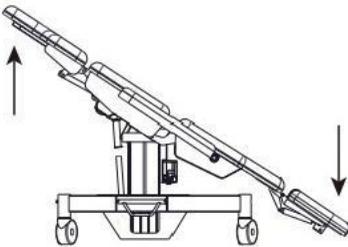


Figure 8

Armrest installation

- Insert the rods on the armrests into the round tubes on both sides of the backrest section. Rotate and fix the pins on the rods into the slots of the round tubes (Figure 11).

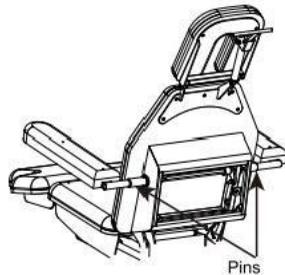


Figure 11

- Press button on line 5 of the hand controller to achieve a one button seated position (Figure 9).



Figure 9

Legend to symbols and information on the product label.

-**Model:** Product identification.

-**Model Num.:** Product version.

- **CE** : CE conformity marking.



: Unique serial number

identifying the product & date
of manufacture.

-**Product Net Weight:** Table weight in kg.

-**INPUT:** Necessary values of the electrical mains supply.

-**ON: 2 min / OFF:18 min.** Indicates the discontinuous operation of the device according to IEC/EN 60601-1, which indicates that 2 minutes of continuous operation of the motor or motors should not be exceeded.



: Consult the instructions for use or the user guide.

- **Max Load:** Maximum load capacity that the table can support.



: Waste from Electrical and Electronic Equipment. Selective waste collection of electrical and electronic equipment is required.



: Classification of medical electrical equipment - Double Insulation.



: Electrical Safety Classification -Type B.

IPX4: Protection against splashing water or liquids.



: Manufacturer identification.

EC REP: Authorized representative in the EU.



: Importer.

TROUBLESHOOTING

If during the life of the table you detect any malfunction or require repairs or maintenance please contact your local dealer and do not use the table until you have received a diagnosis or the table has been professionally repaired.

MAINTENANCE

Carry out a visual inspection at least once a month to check the general condition of the table. Moreover, you must do the following:

-Check the correct adjustment of the different sections.

-Check the proper functioning of the motors and their movements.

-Check and clean the gas struts

-Check and tighten screws (if necessary).

-Lubricate any required parts.

-Check the condition of the electrical wiring and the socket.

- Check the wheel immobilization system.

RECYCLING

If you need to dispose of the table or any of its parts or accessories consult and respect the local laws and regulations applicable to waste management and protection of the environment in your country.

WARRANTY

This warranty shall remain in effect for 5 years (60 months) for the structure and 3 years (36 months) for the base frame and motor from the date of original consumer purchase. An original sales receipt or proof of purchase may be required in order to redeem the warranty. All repairs to the product must be performed by a service center certified by CoinfyCare. Any modifications or repairs performed by unauthorized service providers or groups will void the product warranty. This statement of guarantee prevails over any other statement specified or implied by the seller.

CLEANING AND DISINFECTION

Before cleaning the table unplug it from the mains.

Clean the table after each use.

Dust the table with a soft dry cloth.

To clean the upholstery apply a solution of warm water and a mild detergent (for example neutral soap) with a well wrung soft cloth. Wipe the table over once cleaned.

Disinfect upholstery and other surfaces with a suitable bactericide for this purpose.

Apply the disinfectant on a soft cloth, never directly on the table.

Never use a mixture of disinfectants or a mixture of disinfectant and soap

Wipe the table upholstery over once cleaned & disinfected with a cloth and water only. This will remove any residue from the upholstery.

Avoid the accumulation of moisture on parts of the metal structure as they could rust.

Since we do not know which disinfectant will be used we cannot guarantee that it will not cause damage to the upholstery. Read the instructions and warnings for use of the disinfectant product.

TRANSPORTATION AND STORAGE CONDITION

Transport condition

The products can be transported by general means of transportation. They should be protected from the sun and rain and should not be mixed with toxic, harmful and corrosive materials.

Storage

The packed electric procedure chair should be stored in a dry and well ventilated environment without corrosive gas.

SHUTDOWN PROCEDURE

Adjust the chair to a suitable position to ensure that the patient is safely left, then turn off the power.

Contraindication

Please note that our beds are not sterilized. Please do not use them according to aseptic standards.

CONTACT

Applicant /company name /after sales service unit:

Huangshan Jinfu Medical Instrument Co.Ltd
Domicile / production address: No.17,
Qiliting,Qimen County, Huangshan City,
Anhui Province, China
Tel: 008657187006218
www.coinfycare.com

European legal representative:

LOTUS NLB.V.
Koningin Julianaplein 10,1e Verd 2595AA,
The Hague, Netherlands

