



Mayo Trolley Assembly

Remove all items from the package. Full package contents are pictured below.



Then take the upright and insert the end indicated below into the large hole in the T shaped base then turn upside down holding the pole in place.

Upright



T Shaped Base



Upside Down Base






Then take the screw & the allen key. Insert screw through the base into the upright and tighten into place.

Screw & Allen Key



Tightened Screw

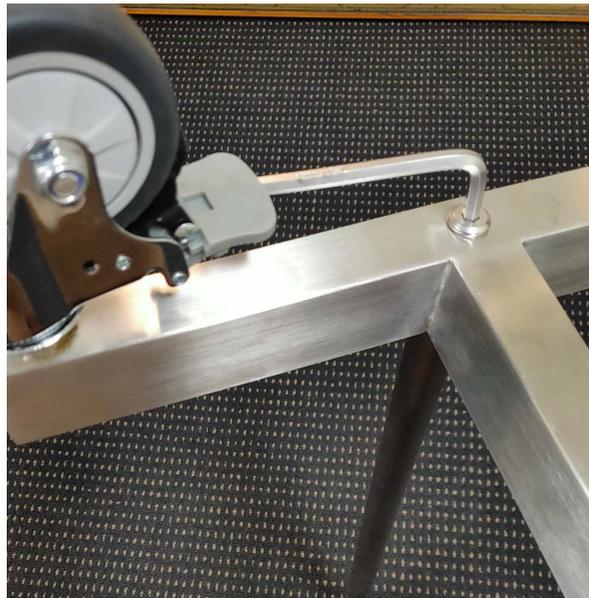


Place the rod with the threaded end and screw into the underside of the rectangular tray frame until all the way in.

Threaded Rod & Tray Frame



Correctly Inserted Rod

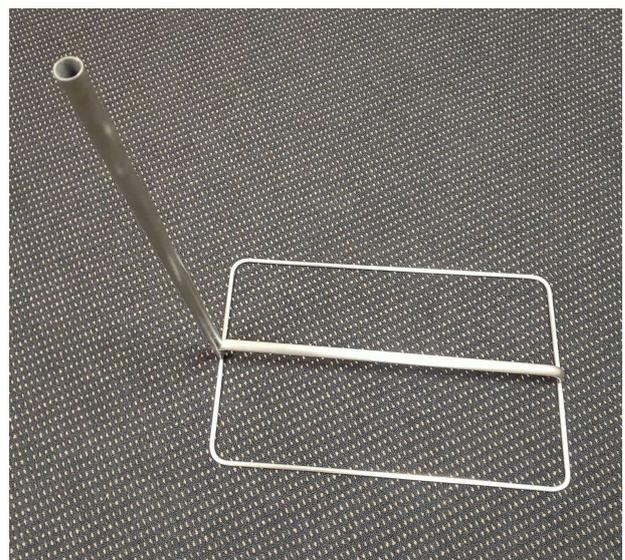



Ensure the base is set upright on the wheels and then insert the tray frame & rod into the upright connected to the base. Then screw the hand wheel into the socket.

Where To Insert



Hand Wheel



Correctly Inserted



Sit the tray in the rectangular frame and assembly is complete.

Correctly Assembled Mayo Trolley

